

## Mini II-Wash Prime Instructions

- Check Water Supply Container to make sure it is filled with de-ionized water.
- Chuck a sample lens and place the Prime select switch to Wash pump prime.
- Press the Cycle start button. The lens will automatically lower over the wash bar and the high pressure wash pump will activate. Let the pump run for approximately sixty seconds to purge all air out of the lines. Flip the Prime select switch back to the center position. The pump should now beat slower and consistently. If the beat does not slow down, flip the Prime select switch back down to Wash prime for twenty seconds, then back to the center position. Repeat as necessary. Press the Cycle stop button when primed.

**Reminder:** Once the Cycle stop button is used, the lens must be unchucked and rechunked in order for the cycle to start again.